[Nom du produit / Product name / Produktname / Nombre del producto / Nome del prodotto]

Masque chirurgical, jetable / Surgical face mask, disposable / Medizinische Medizinische, Einweg / Mascarrillas quirúrgicas, desechables / Maschera chirurgica, monouso

[Norme / Standard] EN 14683:2019+AC:2019 ; Medical Device Regulation 2017/745

[Représentant autorisé européen / EC Authorized representative / EG-Bevollmächtigter / CE representante autorizado / Rappresentante autorizzato CE]

Shanghai International Holding Corp. GmbH (Europe), Eiffestrasse 80, 20537 Hamburg, Germany

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EN

Make sure to wash your hands before putting on the mask. Once in place, do not handle the mask as there is a risk of deterioration of it and contamination of the hands. If using the mask against an infectious risk, practice simple washing or hydro alcoholic rubbing of the hands before and after each change of mask.

1. Hold the respirator by the earloops with the nosepiece at the top and the non-colored fabric side facing inwards.

2. Press the mask against your face with the nosepiece on the bridge of your nose.

3. Place the earloops behind your ears.

4. Pinch the nosepiece with both hands to adjust it to the level of the nose. Lower the bottom of the mask under the chin. Respect the laying direction (nosepiece at the top). The mask should be worn covering the nose, chin and mouth.

After use, remove the mask by grabbing the elastic from the back without touching the front part of the mask. Single-use masks should be discarded immediately after each use in a bin with a plastic bag (preferably with a lid and not manually operated) as it is not possible to decontaminate them. Wash your hands or apply friction with a hydroalcoholic solution after removal.

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### EN

Keep this information for future reference.

This bag contains 10 non-woven 3-ply surgical masks EN 14683:2019 Type IIR for single use. Each mask is non-sterile and consists of a latex-free 3-layer non-woven filtering complex (SMS), a nose bar and 2 ear loops. Surgical masks are Class I, Rule 1 Medical Devices according to Regulation (EU) 2017/745. This mask offers a Bacterial Filtration Efficiency (EFB) greater than or equal to 98% and is resistant to liquid splashes according to standard EN 14683:2019. A surgical mask is a medical device designed to prevent droplets emitted by the person wearing the mask from being projected onto the environment. It also protects the wearer against the projections of droplets emitted by a person opposite. Wearing Type IIR surgical masks is suitable in a medical environment. The correct use of this mask limits the transmission of respiratory viruses. Improper use can cause illness or death.

Save these instructions for future reference. Before use, make sure that the choice of mask used corresponds to the use and level of protection desired. Before use, always check the condition of the mask and its attachments. The mask is intended for single person use. Remove and/or replace the mask immediately if you experience difficulty breathing, nausea, discomfort, dizziness, or other distress or when the mask is damaged/deformed/wet, or proper face fit does not occur. cannot be maintained. Single use mask, should not be used for more than 4 hours. Throw it away after just one use following the attached instructions.

Due to the high contamination of the masks used, it is essential that: the body of the mask is not touched with the fingers / hands of the wearer; hands are disinfected (complete hand disinfection) after removing the mask; a mask covering the nose and mouth of the person wearing it is used and not kept around the neck for reuse; a used mask is discarded when its use is no longer necessary or between two medical procedures; When protection is needed again, a new mask should be worn.

Warning - this product is not flame resistant and must not be used in areas with open flames. This mask does not provide oxygen. Do not use in an explosive atmosphere. Do not use it in low oxygen atmospheres.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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