NOTICE D'INFORMATION FABRICANT / INFORMATION SUPPLIED BY MANUFACTURER / HERSTELLER HANDBUCH / MANUAL DEL FABRICANTE / MANUALE DEL PRODUTTORE / MANUAL DO FABRICANTE
02.10.2023 Version 2

[Nom du produit / Product name / Produktname / Nombre del producto / Nome del prodotto / Productnaam / Nome do Produto]

Gant d'examen nitrile / Nitrile examination glove / Untersuchungshandschuh aus Nitril / Guante de examen de nitrilo / Guanto da esame in nitrile / Nitril onderzoekshandschoen / Luva de exame de nitrilo

[Norme / Standard]

Dispositif Médical de Classe I Règle 1 selon le Règlement (UE) 2017/745 / Medical Devices Class I Rule 1 according to Regulation (EU) 2017/745 / Medizinprodukte der Klasse I Regel 1 gemäß Verordnung (EU) 2017/745 / Dispositivos médicos de clase I Regla 1 según el Reglamento (UE) 2017/745 / Dispositivi medici di Classe I Regola 1 secondo il Regolamento (UE) 2017/745 / Medische hulpmiddelen klasse I regel 1 volgens Verordening (EU) 2017/745 / Dispositivos médicos Classe I Regra 1 de acordo com o Regulamento (UE) 2017/745

EN 455-1:2000, EN 455-2:2015, EN 455-3:2015, EN ISO 13485:2016 ;

EPI de Catégorie III selon le Règlement (UE) 2016/425 / PPE Cat. III Regulation (EU) 2016/425 / PSA Kat. III Verordnung (EU) 2016/425 / PPE Cat. III Regulation (EU) 2016/425 / DPI Cat. III Regulamento (UE) 2016/425 / PBM Cat. III Verordening (EU) 2016/425 / EPI Cat. III Regulamento (UE) 2016/425

EN ISO 374-1:2016+A1:2018, EN ISO 374-5:2016, EN ISO 21420:2020 ;

[Représentant européen autorisé / EC Authorized representative / EG-Bevollmächtigter / CE representante autorizado / Rappresentante autorizzato CE / CE Representante autorizado] Lotus NL B.V. Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands

[Déclaration de conformité UE / EU Declaration of conformity / EU-Konformitätserklärung / Declaración de conformidad de la UE / Dichiarazione di conformità UE / EU-conformiteitsverklaring / Declaração de conformidade UE] Disponible sur / Available at / Verfügbar unter / Disponible en / Disponibile su / Verkrijgbaar bij / Disponível em : www.oneprotek.com

[Organisme notifié / Notified body / notifierte Stelle / Organismo Notificado / Organismo Notificato / Angemelde instantie / Organismo notificado]

SATRA Technology Europe LTD, Bracetown Business Park, Clonee, Dublin D15 YN2P, Ireland. Notified Body number: 2777

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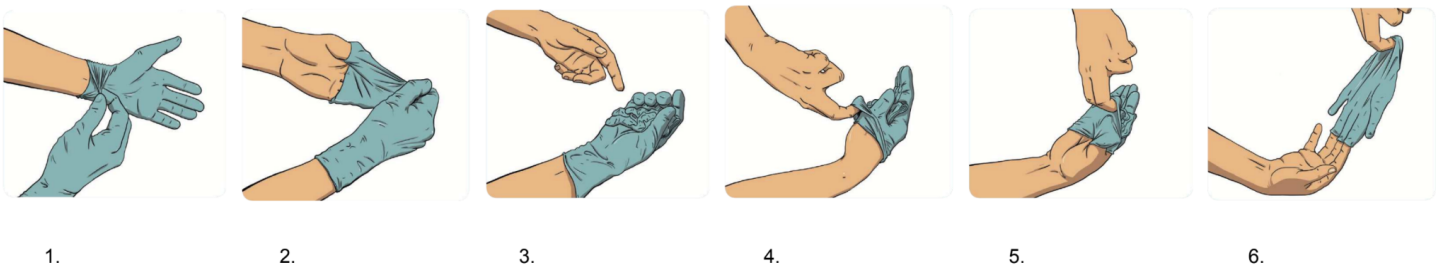
EN

Make sure to wash your hands before putting on the glove. Use with dry, clean and ring-free hands. Use on hands with long or untreated nails and rings could cause the device to tear. The glove must be worn before the start of operational activity. Once in place, do not handle the glove as there is a risk of deterioration of it and contamination of the hands. If using the glove against an infectious risk, practice simple washing or hydro alcoholic rubbing of the hands before and after each change of glove. Contaminated gloves should be cleaned or washed before removals and immediately thrown in a suitable container, in a bin with a plastic bag at a minimum (preferably with a lid and not manually operated).

Steps to remove protection gloves: (1) One gloved hand pinches the glove of the opposite hand a few inches from the edge of the cuff, (2) turn it over and (3) remove it. (4) Then, after inserting a bare finger into the second glove without touching the outside, (5-6) remove it. The two gloves are then thrown into a suitable container and disposed of in a specific disposal circuit for contaminated waste. In case of doubt with the recommended process, refer and proceed with disposal according to the guidelines of the healthcare facility or according to the indications of your local authority.

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EN

Storage between -10°C and 40°C. Keep away from direct sunlight, protect from heat and moisture; store in a cool dry place and keep in the original packaging. Keep away from ozone sources. If gloves are properly stored, as indicated above, they won't lose their performance and won't significantly change the glove characteristics. If gloves could be affected by aging or storage, the expiry date is mentioned on the packaging materials. Shelf life: 5 (five) years for unused gloves from date of manufacture if storage conditions are met.

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EN

Keep this information for future reference. No warranty implied.

Examination glove in nitrile rubber, powder free and non-sterile, ambidextrous, latex free. Gloves do not meet the length requirements of EN 21420 as they are also examination gloves. Components used in the marking gloves may cause allergic reactions in some users. If allergic reactions should occur, discontinue use and then seek medical advice. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Warning - this product is not flame resistant and must not be used in areas with open flames. Do not use it in an explosive atmosphere. Gloves shall not be used for protection against ionizing radiation nor for use in containment enclosures. Gloves not to be used in mechanically aggressive activities and not to be worn when there is a risk of entanglement by moving parts of machines. These gloves are single-use and not designed to be laundered nor to be reused. Do not swallow. Keep out of the reach of children and animals.

Suitable for contact with foodstuffs. Some gloves may show excessive migration towards certain types of foodstuffs. To know which restrictions apply and for which specific foodstuffs the gloves can be used, please refer to the following global migration data sheet or contact OneProtek

Type of foodstuffs - Testing conditions	Aqueous food Analytical tolerance	Alcoholic food Stimulant used: Ethanol 10%	Acidic food Stimulant used: Acetic acid 3%	Fatty food Stimulant used: Rectified olive oil
2 hours/temp. 70°C	<3.0mg/dm2	<3.0mg/dm2	<3.0mg/dm2	<3.0mg/dm2

Protection against chemical risks meeting standard EN 374-1:2016/Type B = chemical breakthrough time > 30 minutes against at least three chemicals (KPT) as per list defined in EN 374-1:2016. Chemical resistance data has been assessed under laboratory conditions from samples taken from the palm only, and only concerns tested chemicals. The data may not reflect the actual duration of protection in the workplace, as gloves can be used in a different mixture or external factors (temperature, abrasion and degradation) than the type test. During use, protective gloves may provide less resistance to the chemical due to changes in physical properties. Snagging, rubbing, movements, degradation caused by chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. Since these gloves are disposable devices, frequent replacement is recommended to ensure that the optimum level of protection is maintained; in any case, the time of use cannot exceed that foreseen by the protection indexes displayed for the different categories of chemical products. Always carry out a preliminary test in real conditions of use. Chemical permeation data test per EN 16523-1:2015 and degradation data test per EN 374-4:2013 are compiled in the below data sheet or available upon request at OneProtek :

Tested chemicals	EN 16523-1:2015 Permeation Level	EN ISO 374-4:2013 Degradation %
40% Sodium Hydroxide (K)	6	-38.4
30% Hydrogen Peroxide (P)	2	17.6
37% Formaldehyde (T)	5	46.6

Breakthrough Time (BTT)	Performance Level
BTT > 10 minutes	Level 1
BTT > 30 minutes	Level 2
BTT > 60 minutes	Level 3
BTT > 120 minutes	Level 4
BTT > 240 minutes	Level 5
BTT > 480 minutes	Level 6

Protection against bacteria, fungi and virus meeting standard EN 374-5:2016. The penetration resistance has been assessed under laboratory conditions and relates only to the test specimens.

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